



K9 SMILES™

Instructions

- 1 Complete Test Order Form (unless completed online)
- 2 Vigorously swab back and forth along entire outer gum lines (between teeth and gums) for at least 30 seconds; test accuracy is dependent on proper collection
- 3 Insert Swab into Collection Vial; bend Swab handle until it breaks at score mark
- 4 Secure cap and place barcode label, lengthwise, on Collection Vial; ensure “Dog Name” is also on the label
- 5 Double bag collected sample using the bag with absorbent pad first
- 6 Place double-bagged sample into the collection box along with the Test Order Form (unless completed online); ensure “Date Collected” and all other required fields are completed
- 7 Close collection box, seal box lid along the adhesive strip, and send via mail
- 8 Samples must be received within 21 days from date of collection

Contact PetDNA Labs™ for assistance:

support@PetDNALabs.com; 855-4DNAPET (436-2738); www.petdnalabs.com

